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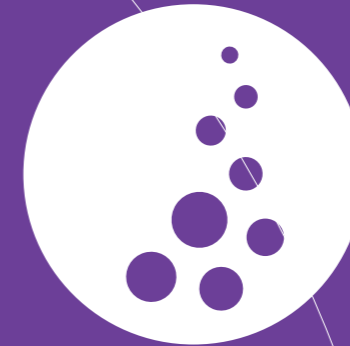
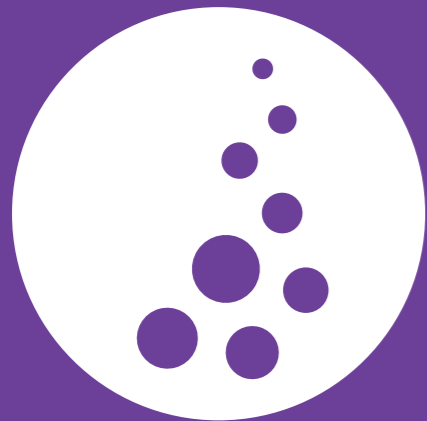
Customer satisfaction is our top priority. Recommendations from satisfied customers are the recipe for our success. All of our machines have medical CE certification and are TÜV certified. We are also authorized to provide training and issue certificates for our devices.

WE ARE READY TO DO OUR VERY BEST FOR YOU

For your salon, we offer the most professional beauty equipment. Our latest-technology machines not only help you increase your customer base, but also enable you to provide the quality they deserve. We represent high quality and excellent service. In addition, we repair faulty devices at a reasonable price. If you have any questions, please contact us via email or call us.

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**CLINICALLY
ENGINEERED**

ADVANCED DEVICES FOR MODERN
AESTHETIC PROFESSIONALS

**GERMAN ENGINEERED
BEAUTY TECHNOLOGY**

ADVANCED DEVICES FOR MODERN
AESTHETIC PROFESSIONALS

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- 03.** Alphasheer PRO Focuslight KI-Technologie
- 04.** Diodenlase Alphasheer PRO 4Wave
- 05.** Alphasheer RF 4Wave
- 06.** Portable Dioden Laser
- 07.** Hydra Beauty 15 in 1
- 08.** Aquastar
- 09.** Aquastar II
- 10.** AlphaSheer Multishape
- 11.** EMS + RF
- 12.** EMS Contouring
- 13.** Cool 360° Cryolipolysis
- 14.** EM Bella
- 15.** Ultra Pulse Fractional CO2 Laser System
- 16.** Picosure
- 17.** Picosecond Nd Yag Lasersystem
- 18.** Portable Picosecond Nd Yag Lasersystem
- 19.** Q-Switch Laser Maschine
- 20.** Micro Needle Matrix RF - With Vacuum
- 21.** BEG HIFU 5D
- 22.** 2 in 1 Ozone Gold Plasma
- 23.** Dermonique Hochwertige Pflegeprodukte
- 24.** Financial Support
- 25.** Wartung & Reparaturen
- 26.** Schulungen, Ausbildung & Nisv.

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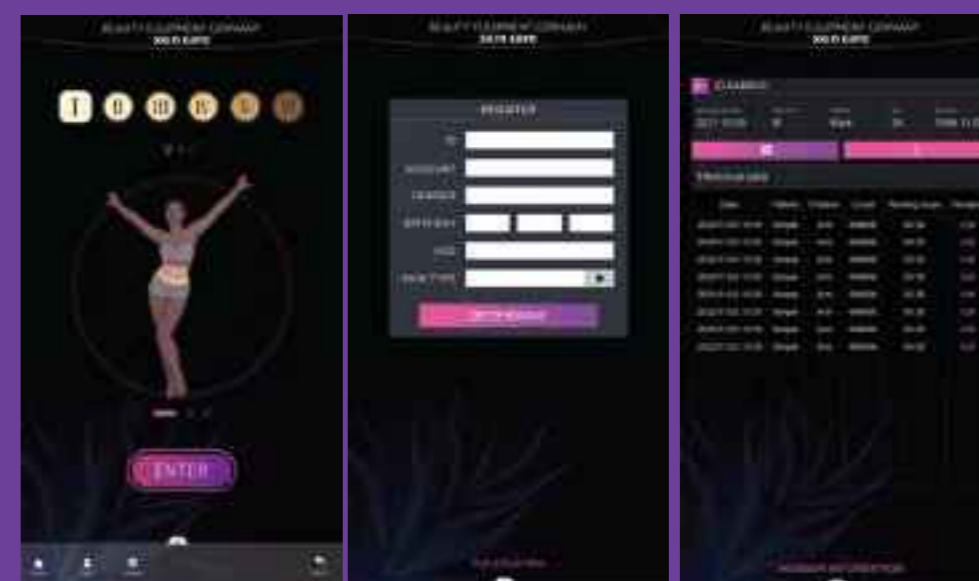
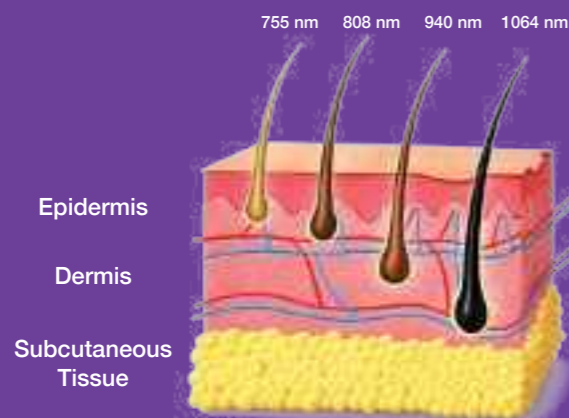
AlphaSheer Focus 4D Teclaser combines advanced 4-wavelength diode laser technology with AI-assisted intelligence to deliver fast, safe, and highly effective hair removal results for all skin types. Its powerful performance, ergonomic handpiece, and large spot size ensure maximum efficiency, comfort, and high return for professional clinics and beauty centers.

TECHNISCHE DATEN TECHNICAL INFORMATION

Lasertyp *Laser Type* : 4-Wavelength Diode Laser with RF Technology

Laser Wellen *Laser Wave* : 755nm/808nm/940nm/1064nm

Kühlsystem *Cooling System* : Contact Cooling system

Leistung *Power* : Professional high-power output

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Available in two colors (Gray & Black) – perfect for small beauty studios or mobile service providers. Very effective thanks to a 3-wavelength laser with 755 nm, 808 nm, and 1064 nm. Powerful cooling system and a laser bar with no shot limit. The transport case makes mobile use easier.

TECHNISCHE DATEN TECHNICAL INFORMATION

Lasertyp *Laser Type* : 3-Wavelength Diode ICE Laser (Optional 4 Waves)

Laser Wellen *Laser Wave* : 755 nm + 808 nm + 1064 nm

Kühlsystem *Cooling System* : Adjustable Water Circulation System

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Hydro-Dermabrasion (Aqua Peeling) is an advanced skincare system that combines multiple innovative technologies in one treatment. It includes vacuum and spray functions, LED light therapy, ultrasound, and high-frequency as well as ion-lifting applications. With a hot/cold handpiece, diamond microdermabrasion, and various tips, the system supports deep cleansing, firming, and skin rejuvenation for optimal results.

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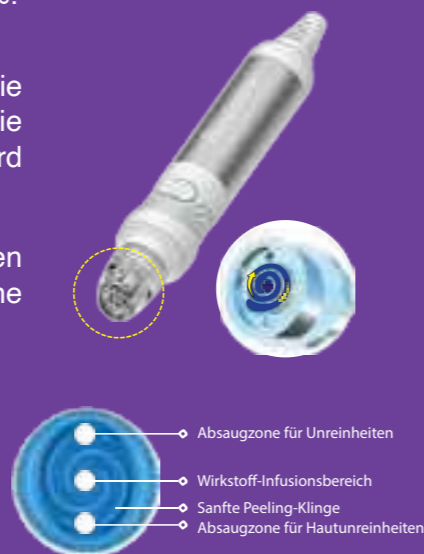
The non-invasive treatment improves skin health, addresses individual concerns, and delivers an unparalleled glow. Unlike traditional facial treatments, it is not performed by hand but with a device, allowing for deeper penetration and longer-lasting results.

TECHNISCHE DATEN TECHNICAL INFORMATION

Behandlungstyp *Treatment Type* : Non-Invasive, Device-Based Facial Treatment

Haupttechnologien *Core Technologies* : RF · Ultraschall · Aqua Peel · EMS / Micro Lift

Wirkung *Benefits* : Deep Cleansing, Skin rejuvenation, Tightening, Long-Lasting Glow

Aqua-Peel System *Aqua Peel System* : Deep Cleansing with Vacuum Technology

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AquaStar II multifunctional skin care device combines advanced Aqua Peel, RF, Ultrasound, and EMS technologies to deliver deep cleansing, skin rejuvenation, and tightening. Its non-invasive system gently purifies pores and enhances the absorption of active ingredients, helping achieve a brighter, smoother, and healthier-looking complexion.

TECHNISCHE DATEN TECHNICAL INFORMATION

Behandlungstyp *Treatment Type* : Non-Invasive, Device-Based Facial Treatment

Haupttechnologien *Core Technologies* : RF · Ultraschall · Aqua Peel · EMS Nano Mesotherapy · Micro Lift · Skin Analyzer

Wirkung *Benefits* : Deep Cleansing, Skin Rejuvenation, Skin Tightening, Whitening & Moisturizing, Long-Lasting Glow

Aqua-Peel System *Aqua Peel System* : Deep Cleansing with Vacuum Technology and Solution Infusion

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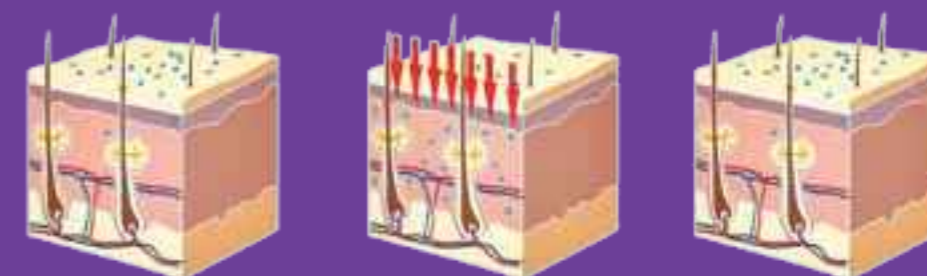
MultiShape is an innovative solution for weight loss and skin tightening. By combining vacuum, RF, and cavitation, the body is sculpted, the skin is firmed, and the appearance of cellulite is visibly reduced. Advanced SuperPulse technology and smooth gliding movements work effectively within the tissue, supporting the reduction of fat deposits and improving elasticity. The treatment is painless, safe, and also suitable for eye and neck lifting.

TECHNISCHE DATEN TECHNICAL INFORMATION

Gerätetyp *Device Type* : Multishape

Bildschirm *Screen* : 15" Touch-Display

RF-Frequenz *RF Frequency* : 1.2MHz Bi-Polar, Multi Polar, Max 50W

Lichttherapie *Light Therapy* : Red LED (630 nm), Blue LED (430 nm)

Kavitation: Frequenz *Cavitation: Frequency* : 1 MHz \pm 10 % Max.

Puls Modus *Pulse Modes* : 4 (Continues , Pulse1 , Pulse2 , Auto)

Leistung *Power* : 50 W

Vakuum *Vacuum*: Max 350mm

Pulsfrequenz *Pulse Frequency* : 1–15 Hz (Continuous, Pulse, Super Pulse)

Lieferumfang *Package Includes* : Machine with 4 Applicators

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HIFEM+RF technology combines high-intensity electromagnetic muscle stimulation with radiofrequency energy to activate muscles at a supramaximal level, improving muscle volume and tightening. At the same time, the high-frequency heat supports the reduction of subcutaneous fat; clinical studies report an average 25% increase in muscle volume and about a 30% decrease in subcutaneous fat.

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HIFEM (High-Intensity Focused Electromagnetic) technology used in EMS makes supramaximal muscle contractions possible contractions that cannot be achieved through traditional training. As a result, muscle volume and density can increase; impulses are intensified and blood circulation in the skin is improved. This method is also known as EMS (Electrical Muscle Stimulation).

TECHNISCHE DATEN TECHNICAL INFORMATION

Gerätetyp *Device Type* : EMS - Muscle Building Stimulation

Pulsfrequenz *Pulse Frequency* : 1–150 Hz

Energiebereich *Energy Range* : 1–100%

Pulsbreite *Pulse Width* : 285 μ s

Magnetische Intensität *Magnetic Intensity* : 0.4–2.5 T (Tesla)

Behandlungsdauer *Treatment Duration* : 20–60 minutes

Lieferumfang *Package Contents* : EMS Device Including 2 Applicators and Straps in 3 Different Sizes

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Cryolipolysis is a non-invasive fat-reduction technology that cools and freezes fat cells under the skin at temperatures between -10°C and +5°C. This process helps prevent premature aging of the fat cells. The fat cells in the treated area are then naturally broken down by the body's metabolism, and the dead cells are eliminated through the lymphatic system over 1–3 months.

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The EMS Chair (EM BELLA) is an innovative and non-invasive solution developed to strengthen the pelvic floor muscles. Thanks to high-intensity focused electromagnetic technology, it stimulates deep muscle tissue; helping to reduce incontinence complaints, support the improvement of prostate problems, and enhance sexual performance.

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The fractional CO₂ laser offers a versatile treatment option for a wide range of applications. It operates at a wavelength of 10,600 nm, which is ideal for stimulating collagen remodeling and makes it an excellent choice for treating a broad spectrum of dermatological concerns.

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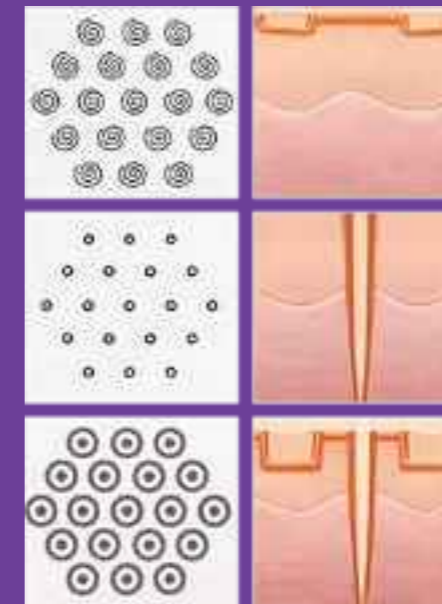
The PicoSure laser machine works on the principle of “light-induced breakdown” and generates ultra-short picosecond pulses that last only trillionths of a second. Because the laser energy is delivered in an extremely short time, it forms very strong pulses that penetrate the epidermis and reach deeper skin layers, breaking ink or pigment particles into tiny fragments, which are then taken up by macrophages and gradually removed.

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532 nm can remove superficial pigments around the eye area, such as moles, nevus of Ota, café-au-lait spots, red tattoos, vascular lesions, and erythema. This wavelength is particularly suitable for darker skin tones. 1064 nm can treat deeper pigmentations on the face and body, such as moles, nevus of Ota, café-au-lait spots, as well as blue and black tattoos.

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For mobile service, the picosecond machine is available in a portable version with the same energy and power. 532 nm removes superficial pigments such as moles, nevus of Ota, café-au-lait spots, red tattoos, vascular lesions, and erythema, and is suitable for darker skin. 1064 nm treats deeper pigmentation, including moles, nevus of Ota, café-au-lait spots, and blue or black tattoos.

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The Q-switch laser uses specific wavelengths that are converted into high energy. This energy is absorbed by the pigments and breaks them into particles. These particles are then shattered into very small fragments, with a large portion bouncing off the skin and other parts being broken down into even smaller particles. Finally, these tiny particles are taken up by phagocytes and eliminated through the lymphatic system.

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Radiofrequency (RF) technology heats the skin's dermis to stimulate collagen production and smooth wrinkles. The skin naturally regenerates and can heal faster after injury. When the temperature reaches 45–60°C, collagen production increases significantly. RF treatment is suitable for deep wrinkles, sagging skin, and acne scars.

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HIFU stands for "High Intensity Focused Ultrasound." It refers to highly focused ultrasound energy that targets not only the dermis but also the upper muscle layer of the skin (SMAS = superficial muscular aponeurotic system), enabling deep and effective tissue tightening.

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The Plasma Pen is an innovative cosmetic treatment designed to improve wrinkles, collagen loss, and sagging skin. It works by ionizing gases in the air to create a tiny electrical arc that does not directly touch the skin but causes controlled micro-injuries on the epidermis. This process, known as fibroblasting, also warms deeper skin layers and may stimulate collagen production for a firmer appearance.

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DERMONIQUE offers the highest standards in skincare with advanced cosmeceutical formulations supported by scientific research. Our products, developed in Germany and successfully clinically tested, are specifically designed for beauty centers and professional treatments, while also integrating seamlessly into personal skincare routines.

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Serums *Serums* : Anti-Aging Serum, Boost Vitamin C Serum, Eye Contour Serum

Toners *Toners* : Alcohol-Based Cleansing Toner, Soothing Alcohol-Free Facial Toner

Reinigungsgels *Cleansers* : Cleansing Milk, Gel Cleanser

Masken *Masks* : Regeneration Night Mask, Detox Facial Mask

Körperpflege *Body Care* : Hydrating Body Lotion, Sunscreen SPF+50

Cremes *Creams* : Deep Moisture Cream, Anti-Spot Cream, Anti-Aging Cream

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LEASING, FINANCING OR RENTING

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
Companies that have been operating for more than 1 year can conveniently lease or use hire purchase. Leasing or hire purchase is also possible without a down payment.

FOR START-UPS, BEAUTY EQUIPMENT GERMANY IS HAPPY TO ASSIST YOU

We prepare a professional offer for your bank. Alternatively, you can finance through us at **0% interest**.

We would be happy to explain the advantages and disadvantages in a personal consultation.

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We offer you a comprehensive service for your devices—before and after purchase. Our team is happy to support you with repairs, rentals, maintenance, filter replacement, or cleaning of our devices. Upon request, we are also happy to provide this service for other devices from external manufacturers.

e.g. maintenance for a diode laser machine incl. filter from €290.00 (plus VAT).

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We provide comprehensive support even after the device purchase. For every situation, you will receive the right answers to your questions, allowing you to relax without any concerns. We support each customer individually and respond to their personal needs our satisfaction is achieved when you are happy.

WARUM UNSER SERVICE? WHY OUR SERVICE?

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Our service stands for fast and reliable support, as well as verified quality through the use of original spare parts. We place great importance on providing personalized support for every customer and benefit from many years of experience in the beauty technology industry.

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Our instructors train you to become a certified cosmetician. As part of this training, everything revolves around facial care and beauty. Treatments such as aquapeeling (HydraSkin), microneedling, BB Glow, etc. are taught comprehensively, both theoretically and practically.

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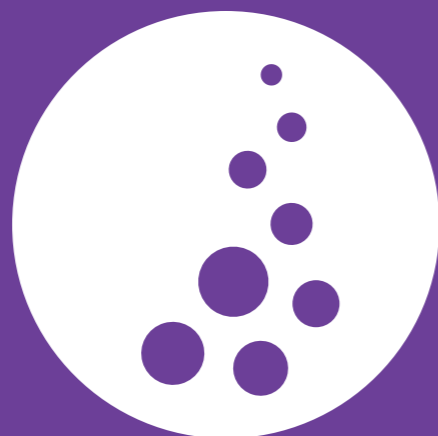
The law of the Federal Ministry for the Environment, Nature Conservation and Nuclear Safety, in force since December 2020, regulates protection against the harmful effects of non-ionizing radiation when used on humans (e.g., IPL, SHR, laser devices, ultrasound).

As Beauty Equipment-Germany, we recommend the METT.NiSV Academy for NiSV training.

BEAUTY
EQUIPMENT-GERMANY
skin care

Beauty Equipment-Germany
Bismarckstr. 4 40822 Mettmann, Germany

Tel.: +49 (0) 2104 77 58 38
Mobile: +49 (0) 173 617 59 17
Mobile: +49 (0) 177 399 66 16

info@beautyequipment-germany.com
www.beautyequipment-germany.de